


LOPEZ VALVE® Directions for Use

To use Lopez Valve

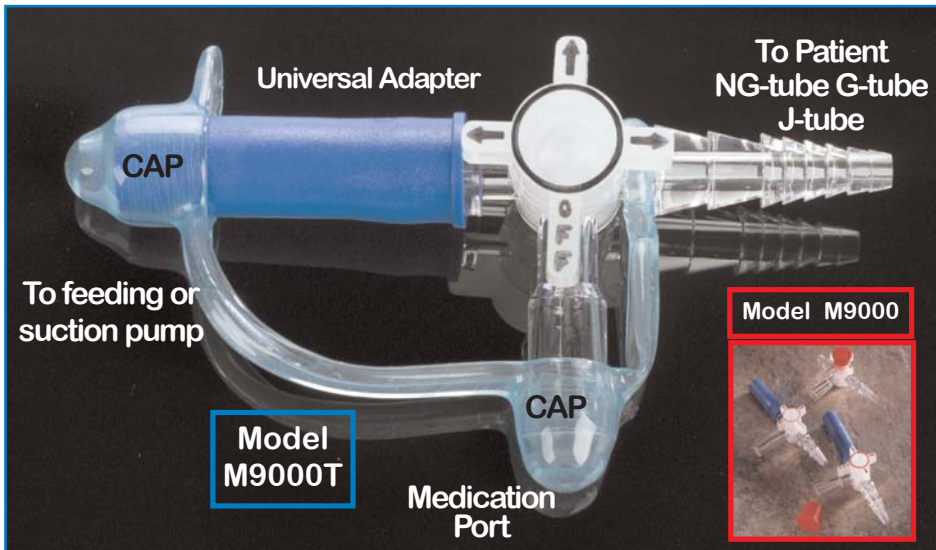


M9000T model: Remove valve from package.

M9000 model: Remove valve, universal adapter and cap from package. Attach “lip” end of adapter to feeding or suction pump port of valve. Push firmly together until secure. Attach cap to medication port.


1. Turn “OFF” indicator to tapered valve port. Attach valve directly into patient’s nasogastric (NG-tube), gastrostomy tube (G-tube), or jujenostomy tube (J-tube). Push firmly together until secure.

Feeding Procedure

1. Remove adapter cap (M9000T). Insert feeding set, pump set, or bolus syringe securely into universal adapter.
2. Turn “OFF” indicator towards the medication port. Arrows show fluid flow from the feeding device to the patient. Begin feeding procedure.
3. When feeding is complete, remove feeding device. Irrigate valve and tube per facility protocol.
4. Turn the “Off” indicator towards the patient. Remove irrigation syringe. Re-cap adapter port (M9000T).

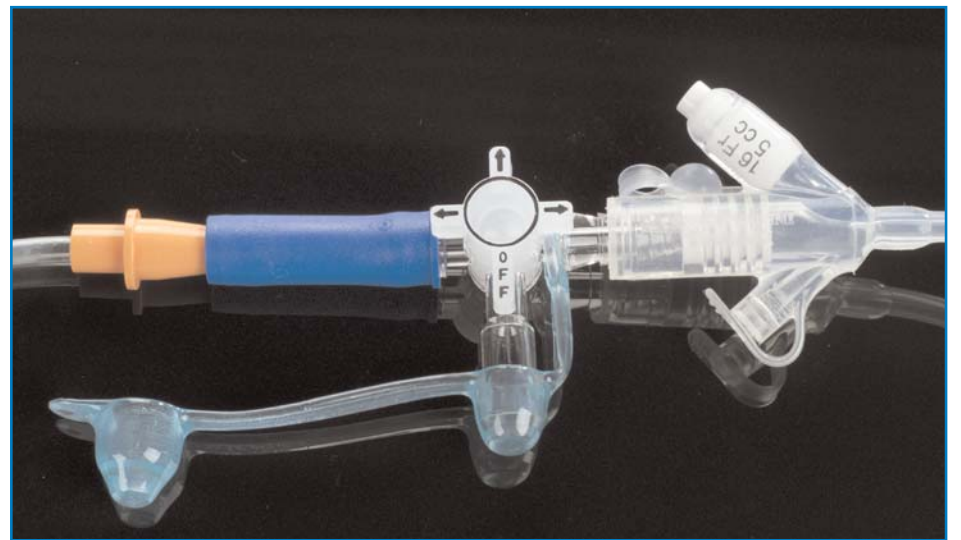

Medication Administration

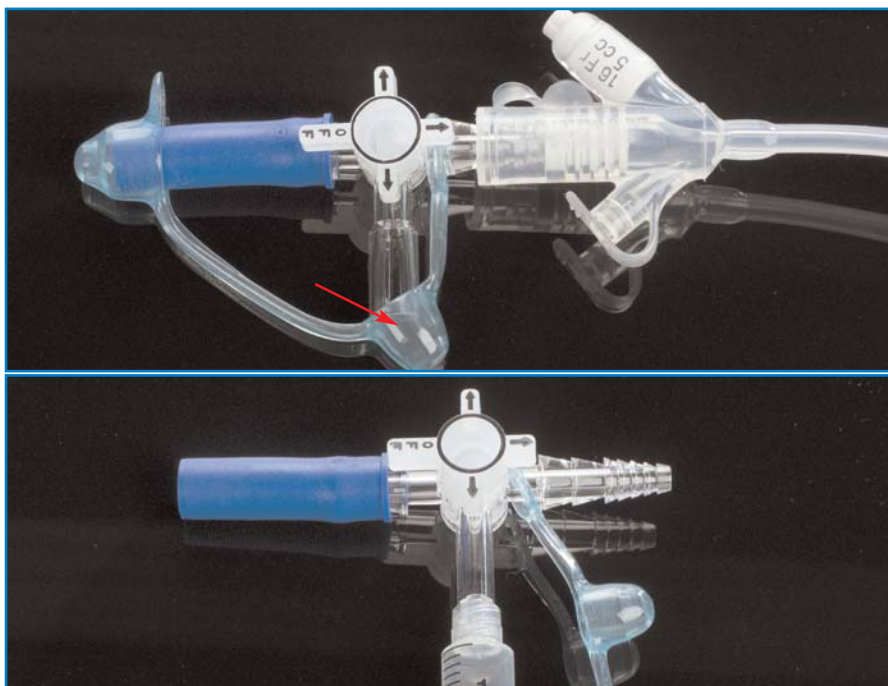

1. Remove cap from medication port. Park cap in turn handle opening of valve.
2. Push and twist catheter tip of administration device into port firmly until secure.
3. Turn “OFF” indicator towards feeding or suction pump port. Arrows will show fluid flow from medication port to patient.
4. Administer medication. Remove medication syringe. Insert irrigation syringe following steps 2 and 3. Irrigate valve and tube per facility protocol.
5. Turn “OFF” indicator towards patient. Remove irrigation syringe. Re-cap medication port.

Suction Procedure

1. Insert suction tubing directly onto suction port or into universal adapter pushing firmly together.
2. Turn “OFF” indicator towards the medication port. The arrows will show suction from the patient to the suction device.
3. Follow suction device directions to initiate and discontinue suction.
4. Turn “OFF” indicator towards the patient and remove suction device.
5. Re-cap feeding or suction port (M9000T).

