



RE: Spiros™ Compatibility with the solvent N,N-Dimethylacetamide commonly found in Busulfex / Busulfan.

Thank you for your interest in the **Spiros™** Closed Male Connector. The **Spiros™** creates a unique needlefree closed system which protects the integrity of the fluid being transferred and /or mixed. The **Spiros™** also meets all of the Personal Protective Equipment (PPE) guidelines for NIOSH, OSHA, ONS and USP.

It should be noted that the **Spiros™** is made of Medical Grade Polycarbonate which is not compatible with a solvent (N, N-Dimethylacetamide) that is used in certain drug preparations such as Busulfex / Busulfan. This is clearly indicated on the Busulfex (Busulfan) package insert. Drug and solvent preparations are continuously changing therefore package inserts should always be reviewed prior to use for compatibility information.

Thank you again for your interest, if there are further questions or concerns please visit our website at www.icumed.com , or contact the corporate offices at (949) 366-2183 or (800) 824-7890.

Technical Services

ICU Medical, Inc.

.