
# Cleo<sup>™</sup> 90 Infusion Set Training Guide

Please see Instructions for Use for a complete list of the indications, contraindications, warnings, and precautions.


## To insert site and connect tubing set





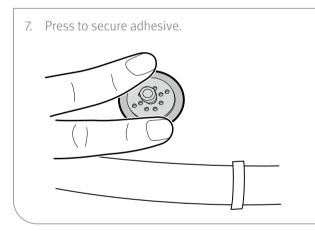


10. Attach tubing to site. The Cleo 90 has eight possible positions.

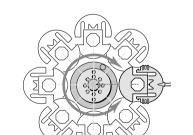




5. Press firmly until you hear a click; hold for 5 seconds; pull straight off.

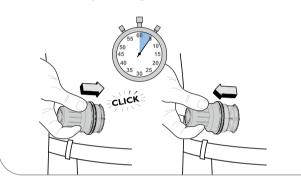


8. Connect tubing and fill according to pump user manual.\*




11. Close the connector; an audible click confirms closure.

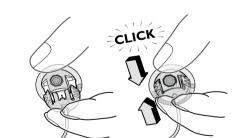

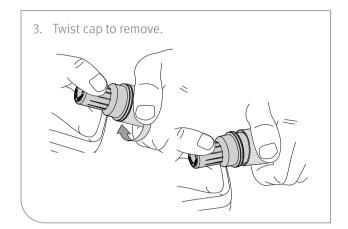




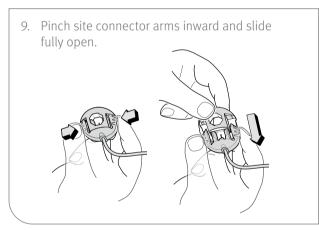

- 12. Fill cannula with medication.
  - > The 6 mm cannula has 0.002 mL priming volume.
  - The 9 mm cannula has 0.003 mL priming volume.




### Disconnection

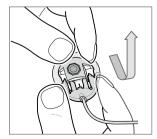

Open the connector by pinching the site connector arms inward and sliding fully open.



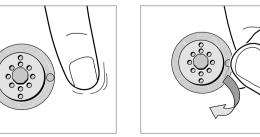

### Changing the Cleo 90 catheter

Change the catheter per your facility policy and procedures.

\* Warning: Never fill the tubing with medication or insulin while the infusion set is connected to your body, or you could deliver an unintended bolus of medication.



© 2023 ICU Medical Inc. | P23-5739

 $R_{\!\!X}Only\,$  For safe and proper use, refer to the Instructions for Use.

800.824.7890 www.icumed.com