



Reference Guide for SwabCap™

Disinfecting Cap for Needlefree Connectors



Peel

- › Remove protective cover by pulling tab.
- › Do not remove SwabCap disinfecting cap from white plastic holder.



Push and Twist

- › Hold needlefree connector in one hand while pushing and twisting the SwabCap disinfecting cap in a clockwise direction onto the needlefree connector.



Protect

- › After the SwabCap disinfecting cap has been twisted into place, gently pull white plastic holder off the needlefree connector.



To Remove for Access to Valve

- › Two-handed removal must be used to make sure the needlefree connector is not removed unintentionally.
- › Grasp the needlefree connector and twist SwabCap disinfecting cap counter clockwise away from the connector. Remove and discard.
- › Inspect the needlefree connector to verify that it's connected securely to the catheter.
- › Connector is ready for access. No further swabbing is necessary after removal, but please refer to your hospital policy.

Precautions: Do not use if package is damaged or opened prior to use. Do not freeze. Do not use if product is dry of alcohol. Use by expiration date on package. Two-handed removal must be used to make sure needlefree connector is not removed unintentionally. Single use only.
Note: Intended to be used and handled by clinical personnel only and care should be exercised to prevent unintended handling by children.

icumedical
human connections

800.824.7890 | www.icumed.com