


Instructions for Use

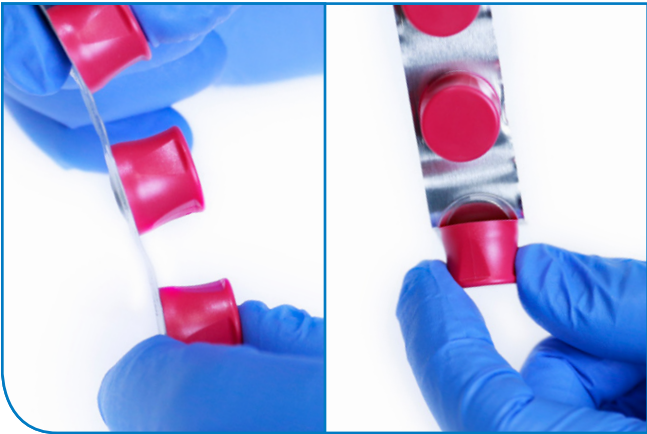

Inspect and Remove

- › Inspect the SwabStrip™ to confirm that cap is sealed to the strip.
- › Grasp the SwabTip and pull off of the strip.



Push and Twist

- › Hold male luer and press and twist the SwabTip onto the luer.
- › Verify that it is securely in place.



Remove

- › To remove SwabTip, grasp the luer and tip and twist SwabTip away from luer until loose. Discard SwabTip after single use—do not reapply to luer.
- › Use aseptic technique to attach disinfected luer to patient line.

Note: Sterile. Use aseptic technique. Single use only. Do not reuse. Do not resterilize. Do not use if package is opened or damaged.

Caution: Do not use if SwabTip is not sealed to strip. Do not freeze. Use by expiration date on package.