



# Product Reference Guide

## Plum 360™ and Plum A+™

ICU Medical Device Cleaning and/or Disinfecting Agents

### Approved Cleaning and/or Disinfecting Agents\*

The following products have been thoroughly tested and validated by ICU Medical and are approved for use with these infusion devices.

	Plum 360 and Plum A+
Cleaning Solution(s)	ASP Enzol™ ASP Cidezyme™ (Outside US)
	Use per manufacturer's recommendations and instructions in the appropriate manual
Disinfecting Solution(s)	Clorox™ Germicidal Bleach (8.25% concentration)
	Do not exceed 1 part bleach in 10 parts water

### \*Alternate Cleaning and/or Disinfecting Agents

These products have only been tested for mechanical and chemical compatibility for ESC (environmental stress cracking). ICU Medical cannot attest to their effectiveness; therefore organizations are encouraged to monitor the effectiveness of these products as facility infection prevention protocols are followed.

- › EcoLab OxyCide™
- › Wexford Labs Wex-Cide 128
- › Johnson & Johnson/ASP® CIDEX® OPA
- › PDI Sani-Cloth® Bleach Germicidal Disposable Wipes
- › Clorox Healthcare® Bleach Germicidal Wipes
- › Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes
- › Clorox Dispatch® Hospital Cleaner Disinfectant Towels with Bleach
- › Diversey Avert® Sporicidal Disinfectant Cleaner Wipes
- › Diversey Oxivir® TB Wipes
- › Cambridge Sensors Microdot® Bleach Wipes
- › PDI Sani-Cloth® Detergent Wipes (OUS)
- › PDI Sani-Cloth® Chlor Wipes (OUS)
- › EcoLab Healthcare Incidin™ OxyWipe S (OUS)
- › GAMA Healthcare Clinell® Detergent Wipes (OUS)
- › GAMA Healthcare Clinell® Clorox Wipes (OUS)

### Do Not Use

- › Accel® Disinfectant TB Wipes
- › PDI Sani-Cloth® AF3 Germicidal Disposable Wipe
- › PDI Sani-Cloth® Plus Germicidal Disposable Cloth
- › PDI Super Sani-Cloth® Germicidal Disposable Wipe
- › Cidex® glutaraldehyde based disinfectant
- › Methyl Ethyl Ketone (MEK)
- › Virex® Organic Ammonium Chloride based disinfectant
- › Ethanol (Ethyl Alcohol)
- › 70% Isopropanol (isopropyl alcohol or IPA)
- › Dimethyl Benzyl Ammonium Chloride
- › Didecyl Dimethyl Ammonium Chloride
- › Clinell Universal Sanitising Wipes
- › Cleaning or disinfecting products not listed as approved for use.

**⚠ CAUTION:** Use all agents as specified by the manufacturer, to avoid infuser damage. Use only recommended agents and follow manufacturer's recommendations regarding dilution. Using agents not recommended by ICU Medical may result in device damage.

Products not listed above as an approved or alternate cleaning and/or disinfecting agent have not been approved for use on the device.

\*Disinfecting properties of cleaning solutions vary, and not all cleaning solutions are disinfectants. Check product labeling or consult the manufacturer for specific information.

Instructions on how to clean and disinfect the infusion device for reprocessing can be found in the associated Technical Service Manual.

You may also call the ICU Medical Technical Support Center at 1-800-241-4002, or visit [www.icumed.com](http://www.icumed.com) for further information.

℞ Only. For safe and proper use, refer to the appropriate manual, guide, or instructions for use.

Reference: IFU0000141 (01, 2020-05)\_Plum 360 Technical Service Manual

© 2021 ICU Medical Inc. | US-ED04-0063-1021R5 | For Distribution

**icumedical**  
human connections

800.824.7890 | [www.icumed.com](http://www.icumed.com)